MOPEDIA®

MA RP881_RP888_01A_05 2016

**IMBRACATURE
MANUALE D'ISTRUZIONE**

**SLINGS
INSTRUCTION MANUAL**

INDICE

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I Class Medical Device
according to Directive 93/42/EEC and further modifications

RP881 Standard sling with headrest

RP888 Standard sling with reinforced headrest

1. INTRODUCTION

Thank you for purchasing a MOPEDIA sling by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

In case of doubts, we recommend to contact the dealer who can help and advice you properly.

2. INTENDED USE

The MOPEDIA by MORETTI slings are designed to transport patient with mobility disease.

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

3. EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the antidecubitus cushions category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question ARE NOT MEASUREMENT DEVICES
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION
4. The devices in question are sold in NON STERILE packing
5. The devices in question should be considered as belonging to class I
6. DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications
8. Moretti S.p.A. declares that the fabric is flame retardant and respects the EN 1021-1 and EN1021-2

4. GENERAL WARNING

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the sling
- Disinfect the item after every usage
- SERVICE LIFE- the sling use limit is defined by the wear of the parts


- Carefully read the patient lift or stand up lifter manual and follow the instruction suggested on the same
- In case of use of harnesses with a different maximum load from the one provided for the lift or stand up lifter, always use the lowest maximum load.

5. GENERAL DESCRIPTION


Sling for all kind of patient lift. Made in polyester (100%), especially suitable for water contact, strong and breathable, sewn with polyester thread. The border is made by a strong ribbon (100% nylon), which make three eyelets in every extremity to guarantee a perfect balance. The belt continuous and swen around the entire perimeter of the sling to ensure greater toughness. The seams are made using high strength polyester yarn.

The RP888 model has the headrest reinforced to increase the head support making the transfer more comfortable.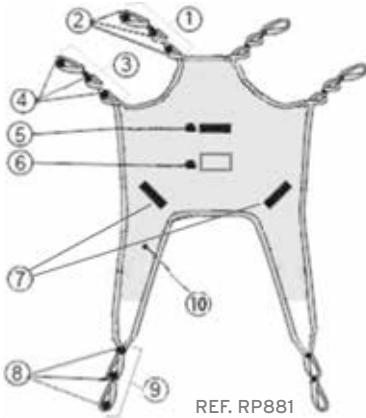

- 1. Upper eyelet
- 2. Loops
- 3. Intermediate eyelet
- 4. Loops
- 5. Movement handle
- 6. Label
- 7. Movement handle
- 8. Lower eyelet
- 9. Loops

REF. RP881

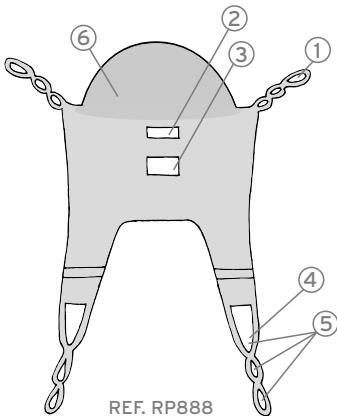- 1. Upper eyelet
- 2. Movement handle
- 3. Label
- 4. Lower eyelet
- 5. Loops
- 6. Reinforced headrest

REF. RP888

6. BEFORE USE

Always verify the sling is in perfect condition, carefully check the seams and the loops.

7. USING WARNING

- Before use, make sure the device is not damaged: the tissue, the border, the sewing and suspension loops must be intact
- Periodically check for any damage or usury, every 6 month at least, in particular if you often clean the sling
- Ensure the loops are correctly hooked to the cradle
- Store and use away from flame or source of heat
- Do not exceed the maximum load

- Evaluate the risk, in order to guarantee the exact body support unit for the patient, according to the correct dimensions, typology and shape for him
- In case of use of sling with a different maximum load from the one provided for the lift or stand up lifter, always use the lowest maximum load

8. HOW TO USE

The MOPEDIA slings by Moretti can fit all kind of patient lift, fixed and movable.

Fig.1

Fig.2

Fig.3

Fig.4

8.1 From a sitting position

1. Ensure that the straps face outward. Pull the sling down until the user's back leaving the opening of the sling at the base of the spine.
2. Check that the sling is aligned properly at shoulder height. (Fig.1)
3. Lift the patient's leg and slip the strap under it and pull it up between their legs, ensuring that the sling is not creased under the thigh. (Fig.2)
4. Repeat the procedure with the other leg.
5. Cross the straps over the legs and attach them to the front hooks on the hoist spreader bar.
6. Raise the hoist slightly and attach the shorter, shoulder straps to the spreader bar. Move the hoist towards the patient while doing this. (Fig.3)

ATTENTION! The long straps are for the legs, and the short ones for the shoulders

8.2 From a lying position

1. Roll the patient aside and position the sling on the bed (Fig.4) (the sling has to be smooth without any wrinkle and it is aligned properly at shoulder height)
2. Make sure that the opening at the bottom of the sling is positioned at the base of the spinal column.
3. Roll the patient on his/her back again (supine position)
4. Cross the bands under the patient's legs and adjust the sling as shown for the sitting position (Fig.2), maintaining the shoulders's bands as short as possible.
5. You are now ready to hang the straps to the cradle (Fig.3).

NOTE: Moretti recommends that two assistants (one on each side of the bed) be used when positioning the patient in a sling.

ATTENTION! Ensure that the patient's head is supported at all times

9. MAINTENANCE

The devices MOPEDIA by MORETTI are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 6 months.

10. CLEANING AND DISINFECTION

Machine wash with a gentle cycle, 30°C

- Do not Bleach
- Do not dry clean
- Dry Flat away from direct sunlight
- Tumble dry low
- Do not iron
- Use non-enzyme detergent
- Do not wash with other colors

11. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the sling in the appropriate disposal areas for recycling.

12. TECHNICAL FEATURES

REF. RP881



MATERIAL 100% Polyester and Webbing 100% nylon (Flame retardant fabric EN 1021)
 MAX WEIGHT 180 Kg

Size	A	B	C	D
ONE SIZE	143 cm	80 cm	93 cm	35 cm

REF. RP888

MATERIAL 100% Polyester and Webbing 100% Polyester (Flame retardant fabric EN 1021)
 MAX WEIGHT 180 Kg

Size	A	B	C	D
ONE SIZE	120 cm	80 cm	70 cm	70 cm

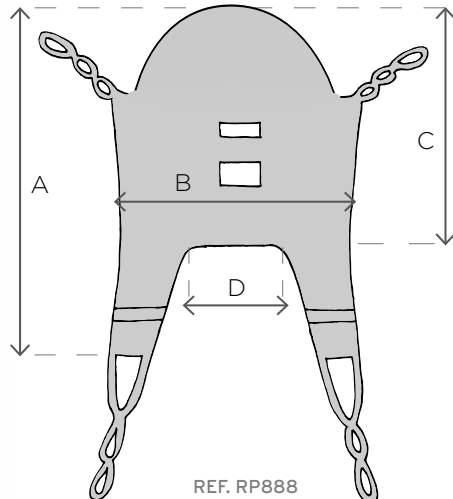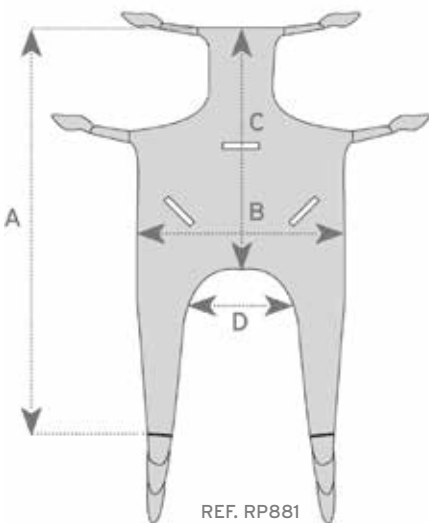13. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

14. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**WARRANTY CERTIFICATE**

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: **info@morettispa.com****MADE IN P.R.C.**

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com